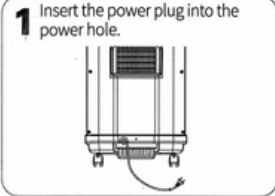

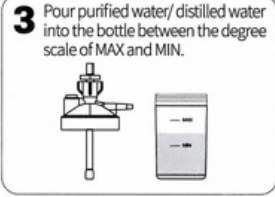



INSTRUKCJA KONCENTRATOR TLENU OXYFLOW


- 1** Insert the power plug into the power hole.

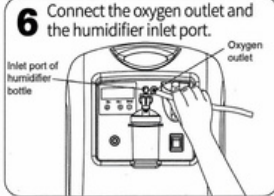

- 2** Take out the humidifier bottle and open the bottle cap.



- 3** Pour purified water/ distilled water into the bottle between the degree scale of MAX and MIN.

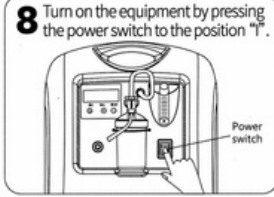

- 4** Cover the bottle cap.

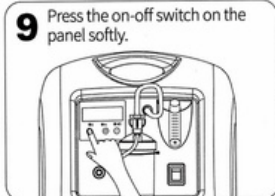

- 5** Fix the humidifier bottle back to the right place.



- 6** Connect the oxygen outlet and the humidifier inlet port.



- 7** Take out the nasal oxygen tube, and connect it with the oxygen outlet port of humidifier bottle.


- 8** Turn on the equipment by pressing the power switch to the position "I".


- 9** Press the on-off switch on the panel softly.


- 10** Twist the knob to adjust the flow rate.



- 11** Set up the time (press the time button first, then adjust the time with "+", "-", "*")

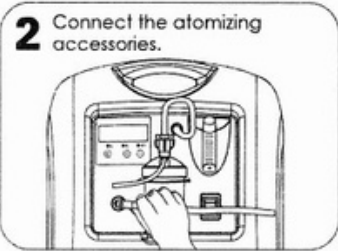

- 12** Warm tips

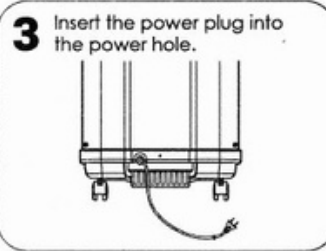
 1. Please operate the machine according to the correct steps.
 2. Do not operate the machine with wet hand in case of the electric shock.

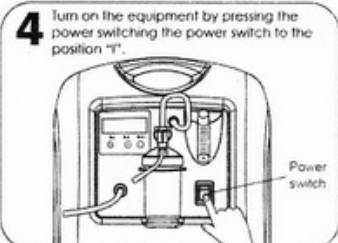
Atomization

- 1** Remove the end cap of atomizing port.


- 2** Connect the atomizing accessories.


- 3** Insert the power plug into the power hole.


- 4** Turn on the equipment by pressing the power switching the power switch to the position "I".


- 5** Press the on-off switch softly on the panel.

